

# 1. Prepare to deploy your Westcot



2. Unbuckle Closure Strap and lay Westcot flat on back.




3. Erect all three legs and account for all Safety Rails, Upper, and Lower IV Pole Pieces.






4. Remove mattress from underbelly and engage all six Leg Locks.

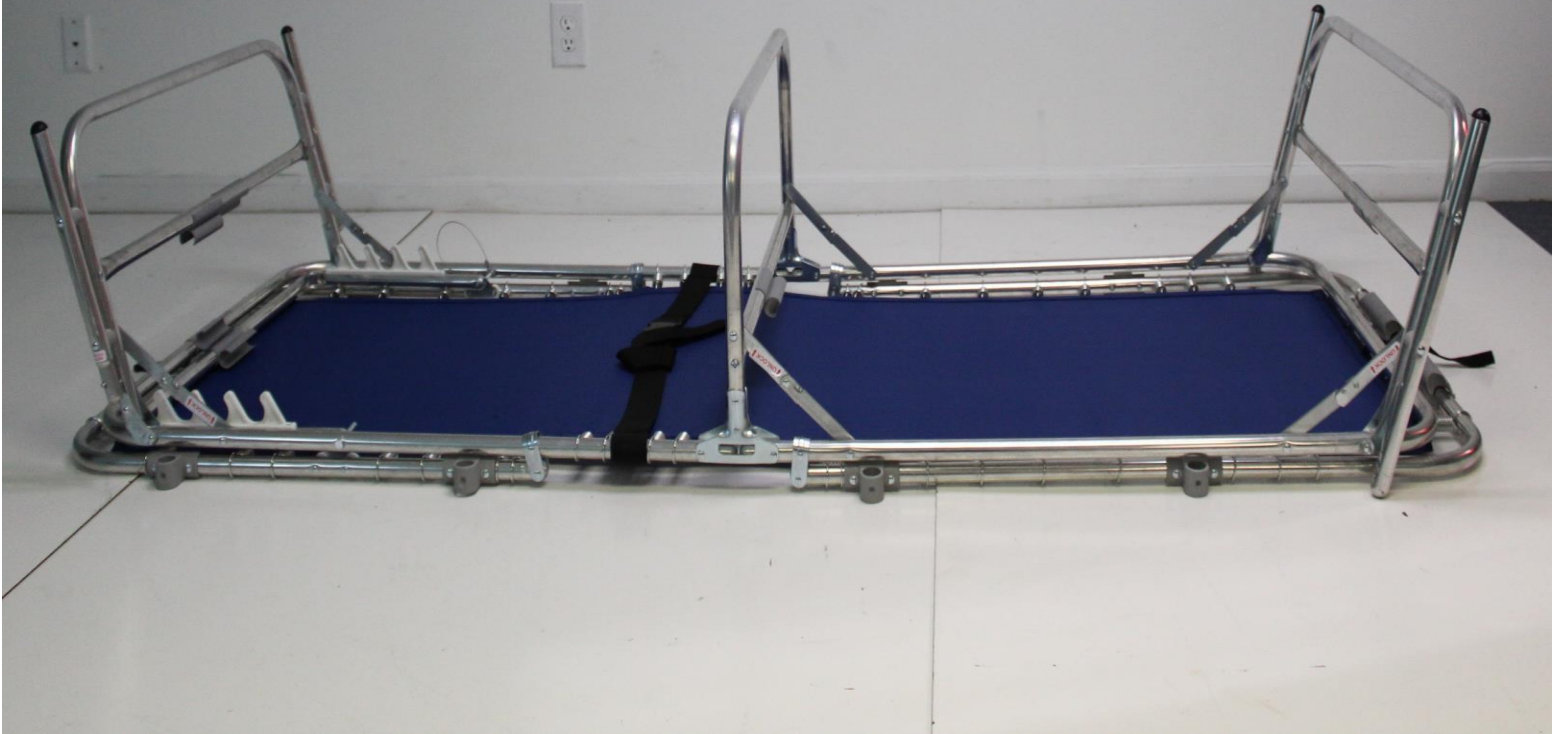

5. Rotate Westcot to the standing position, dress the Mattress across the Decking Platform, and secure Mattress to frame with the Velcro Straps.

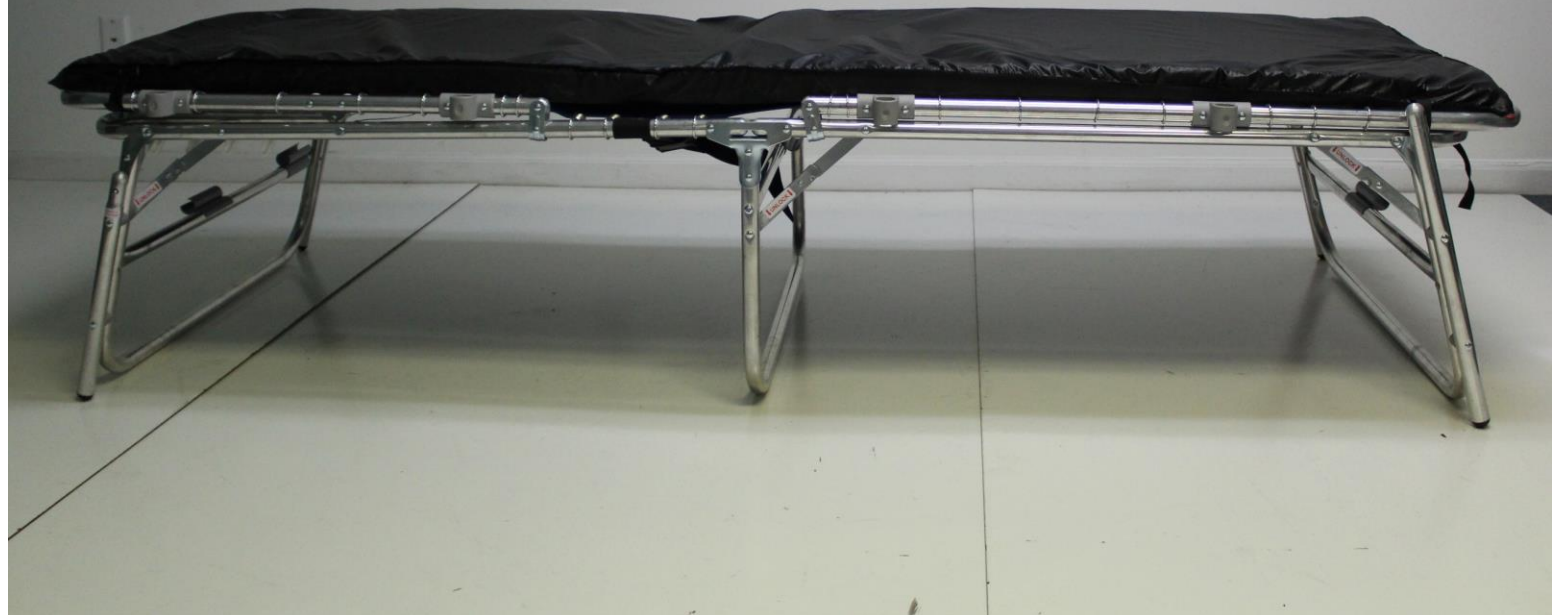

6. Using the Push Button Retainers; insert the Safety Rails and the Two-Piece IV Pole into their respective Mounting Brackets.



7. When using the Head Elevator be sure to engage the Head Elevator Locking Cord (Red) for safety and quick adjustments.





8. To use the Foot Elevator – lift Foot End Frame and extend the Foot Elevators to their lowest position before resting them on the Lower Frame of Westcot.

