

1. Prepare to deploy your Westcot Shelter in Place Cot.




2. Unbuckle Closure Strap and lay Westcot flat on back.

*Note, you will not see stow on board Safety Rails and IV Pole for this model.



3. Erect all 3 Legs, Remove Antimicrobial Soil Resistant Mattress, and engage all 6 Leg Locks.

* Note, if contamination is a concern, perform this step with product on its' side.

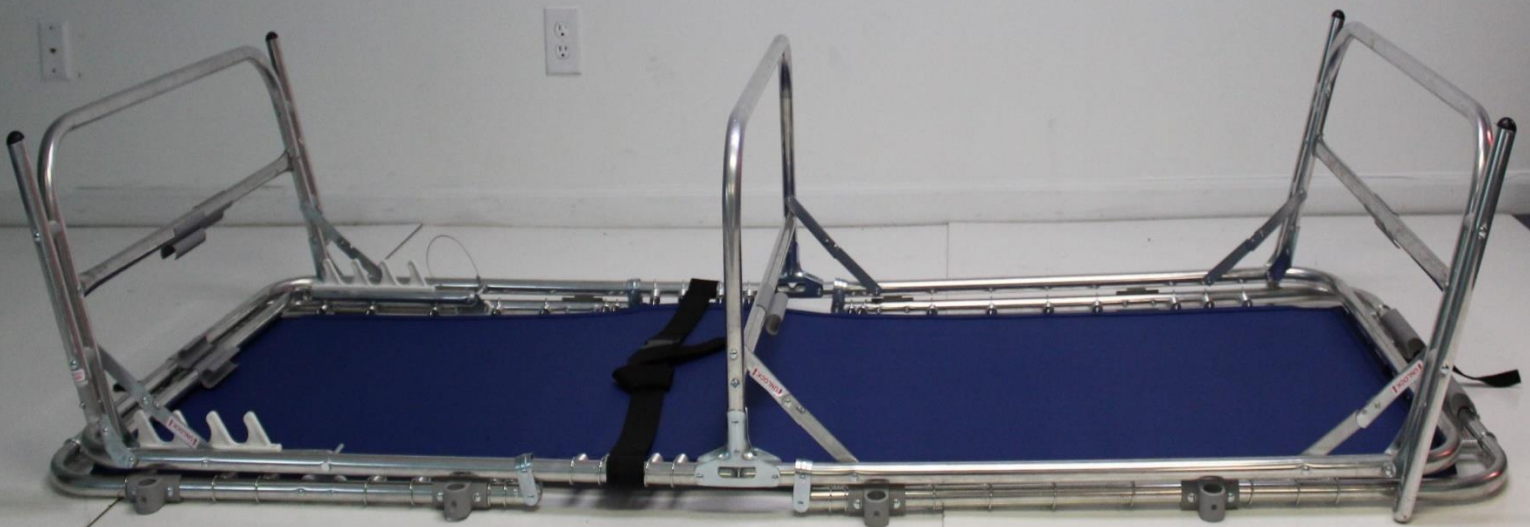

4. Rotate Westcot to the standing position, dress the Mattress across the Decking Platform, and secure Mattress to frame with the Velcro Straps at all four corners.



5. When using the Head Elevator be sure to engage the Head Elevator Locking Cord (Red) for safety and quick adjustments.



6. Sleep/Rest comfortably while awaiting mission assignments.



For More Information:


855.505.9267

IMS4BEDS.COM

SALES@INTEGRITYMEDICALSOLUTIONS.COM



MADE IN

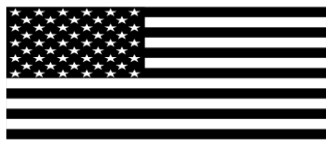

U. S. A.